



Walmay Medical
PRODUCTS

UNIVERSAL LIQUID NITROGEN EXTRACTOR

MODEL NO: 32-NE21

Ensure safety and reliability when working with liquid nitrogen with the Walmay Liquid Nitrogen Extractor.

- Double insulated handle for thermal protection.
- Extendable siphon tube and coned bung makes it suitable for most 10 - 30 litre Dewars.
- The see-through plastic protector disc covering the bung allows accurate insertion/withdrawal from the dewar.

**SUITABLE
FOR MOST
10 - 30 LITRE
DEWARs**



FOR FILLING THE WALMAY CRYOSPRAY

Insert the leg into the Dewar until the bung seals off the mouth opening and the boiling nitrogen pushes it's way up the extractor leg and flows out the handle. To stop the flow, remove the extractor out of the Dewar.



For more information phone **02 9432 2899** or email sales@cryospray.com.au